


Malvern Medical Developments Limited

Instructions For Use – S10 ORACOL Saliva Collection System


1. This sponge swab is designed to be used like a toothbrush. Firstly open the package and remove the tube.

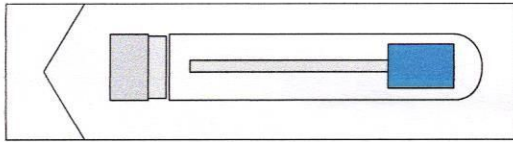

2. Saliva is collected by rubbing the sponge firmly along the gum; (at the base of the teeth if present) until the sponge swab is wet, this takes about one minute.



3. Replace the swab in the test tube and insert the cap. Return sample to the laboratory for processing.

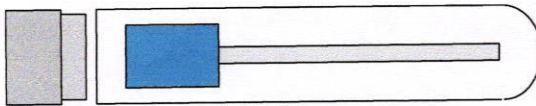

Unit 10, Northbrook Close, Worcester Worcs. U.K. WR3 8BP
Telephone: +44 (0)1905 731343 Fax: +44 (0)1905 731344